aycan workstation

Quick Install Guide
Manager . Plug-ins
License Server (optional)

Software
Menu/Menü
Help/Hilfe/?

aycan Digitalsysteme GmbH
Innere Aumühlstr. 5
97076 Würzburg . Germany

CE 0297

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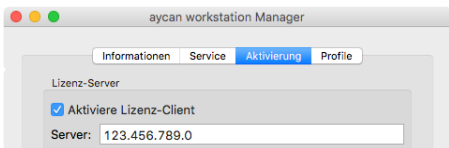
Preparation	13
Installation program	13
Application folder . Dock	14
Manager . Information / Plug-Ins	14
Manager . Service.....	16
Manager . Activation / Activation Codes.....	16
Manager . Profiles	17
Manager . License Server (optional) / Configuration.....	18
Manager . License Server (optional) / Active Licenses.....	19
Manager . Activation / License Server (optional)	19

© 2005-2023 aycan Digitalsysteme GmbH. All Rights Reserved.

aycan Digitalsysteme GmbH
Innere Aumühlstr. 5 . 97076 Würzburg . Germany
info@aycan.com . <https://www.aycan.de/ueber-uns/impressum.html>

aycan workstation Quick Install Guide version 4.00.000

15th edition English – 2023-05

Valid for Software **aycan workstation** as of v4.00.000.

For safe, sound and reliable usage of the **aycan workstation** software, the software must only be used in conjunction with a version of the Apple macOS operating system, which has been approved by aycan Digitalsystem GmbH.

The operating system software version for which an **aycan workstation** release has been approved can be found within the documentation for that version. Documentation is included with each Installation package.

General information about which software versions are approved with which Apple macOS operating system versions can also be found at:

<https://www.aycan.de/produkte/aycan-workstation/faq.html>

Minimal system requirements for **aycan workstation** systems:

Hardware: Apple Mac with Intel processor or Apple Silicon

RAM: 8 GB (minimum; 16 GB recommended)

Hard drive space (SSD recommended): 10 GB

External DVD drive required for CD or DVD read or write access

Operating system: Apple macOS; compatible versions:

see <https://www.aycan.de/produkte/aycan-workstation/faq.html>

Depending on usage, a faster processor, more RAM, and a larger hard drive are recommended.

Symbols used:

Observe electronic instructions for use:

Prior to using the software, all users are obligated to carefully read through the user documentation in the language they understand and to familiarize themselves with its operation. The instructions for use are available in electronic form in the software itself:
Menu 'Help' > 'aycan workstation manual'.

Date of manufacture:

Year of manufacture of the present software version.

Manufacturer:

Name and address of the manufacturer of the software – also in combination with the „Date of manufacture“ (see above).

MD label:

Indicates that the product is a medical product.

CE label:

Through the use of the CE label, the manufacturer declares that, in accordance with EC Regulation 765/2008, the product conforms „with the applicable requirements of the relevant [European] Community harmonisation legislation providing for its affixing.“

UDI label:

Unique Device Identification (UDI) is a worldwide system for uniform product identification for medical devices. The manufacturer identifies the product in the About Dialog/Splash Screen.

Preparation

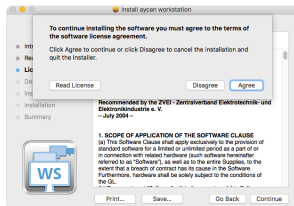
Ensure that **English** is the preferred language on top of the list in the Apple macOS System Preferences > Language & Region.

Plug the **dongle** into a free USB-port.

Start the installation by double-clicking on the installation package.

Installation program

Click on "Continue" and you will be asked to read the software license agreement. If you agree, please click "Agree" first and in the following dialog on "Continue" and finally on "Install".

Application folder . Dock

Locate the **aycan workstation Manager** in the “Applications” folder. Place the application at your preferred location in the dock via drag & drop. There, you will always have immediate access.

Manager . Information / Plug-Ins

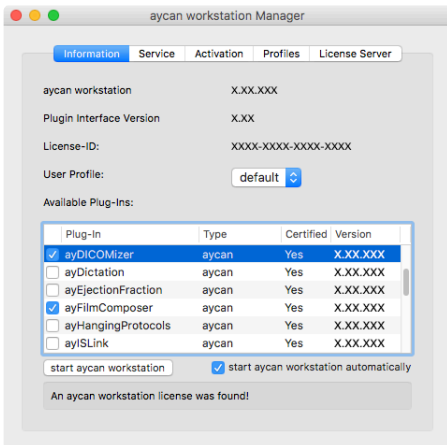
Only a plug-in that is labeled **Yes** is part of the **aycan workstation** software having the CE mark and FDA clearance. Any other plug-ins that may be installed, however, have **No** as the certification status.

Manager . Information / Plug-Ins

When starting the **aycan workstation Manager** for the first time, you will be asked to read the license agreement. If you agree with it, please click "Agree". By clicking the tab "**Information**" you will get a listing with the versions of the program components as well as the 4x4-digit License-ID of your attached dongle. "User Profile" shows a selection list of defined profiles (See "Profiles" tab for details).

Underneath, there is a summary of available **Plug-Ins**. By using the checkbox you can comfortably activate and deactivate the specific Plug-Ins. The columns can be sorted by clicking on their heading and be rearranged via drag & drop. You can restart the program at any time by clicking on "**start aycan workstation**" (alternative: cmd-R).


No valid license found. aycan workstation will quit in 110 seconds. Please save your data.

If no dongle is in place, or the dongle is being removed, a message will appear that **aycan workstation** will shut down after 2 minutes.

Manager . Service

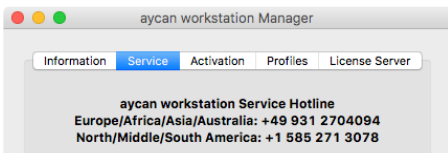In case of any problems, you can contact the aycan service team by clicking on the “Service” tab.


Manager . Activation/Activation Codes

In case that you do not have a Dongle (e.g. in the 30-day demo version), the following message will appear in the lower area of the Manager:


Could not find a Dongle!

Please wait until the info window “No valid license found” appears. If a valid dongle is connected with the system, then click on “Start compatibility mode.” Please contact aycan as soon as possible to solve the compatibility issues.

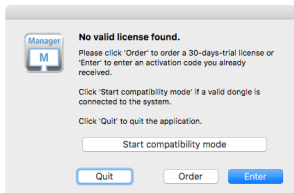To retrieve an activation code, click on “Order” and send the e-mail that has previously been filled in together with your contact data to info@aycan.de.


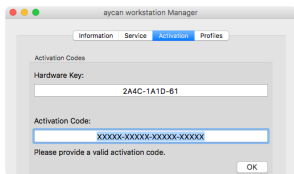
After receiving the activation code from aycan, enter it by clicking on “Enter” in the according field at the tab “Activation”. After confirming with “OK” you can start **aycan workstation**.

Manager . Profiles

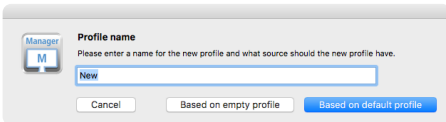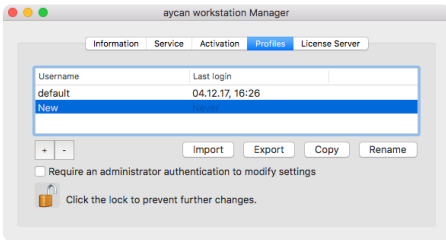In the “Profiles” tab you can create and manage multiple profiles if different users want to use the same workstation with different configurations. Select profiles in the “Information” tab.


To safeguard profiles from unwanted changes, you first need to lift the protection by clicking the lock under the “Require an ‘Administrator’ authentication to modify preferences settings” option. The “Default” profile is created by default and cannot be deleted. Use “+” to create a new profile. Enter a name and a source. Under “Based on empty profile”, all settings are at the level of a new installation; “Based on default profile” copies the most recent settings of the standard profile.

Other options: “-” deletes the selected profile from the list on request; “Copy” creates a duplicate of a profile, “Import” and “Export” load or save the profile list between different workstations. Double-click on an existing profile to change its name – or click on “Rename”.

Manager . License Server (optional) / Configuration

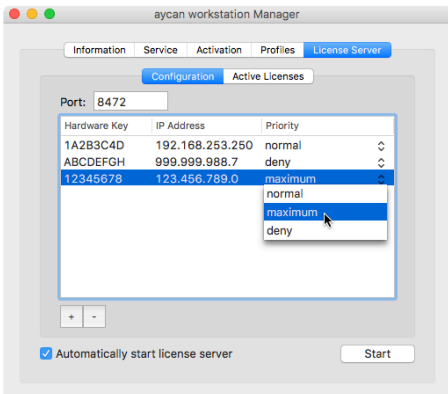In the tab "Configuration" of the License Server (optional), please enter the **aycan workstation** which you want to activate as well as their authorization.


First of all you need to define the "Port" at which the server service shall communicate. Click the "+"-symbol in order to add a new **aycan workstation** to the list. Enter the according "Hardware Key" and the "IP Adress". There are three "Priority" options: "maximum" ensures a steady authorization which is reserved for the workstation permanently. One priority level below is "normal". This will provide authorization on the principle of "first come, first serve" as long as there are licenses according to "maximum". With "deny", you refuse any requests, the workstation will not get a license. With the "-"-Symbol you remove entries from the list.

Start the license server by clicking on "Start". "Automatically start license server" automates this process when starting the **aycan workstation Manager**.

Manager . License Server (optional)/Active Licenses

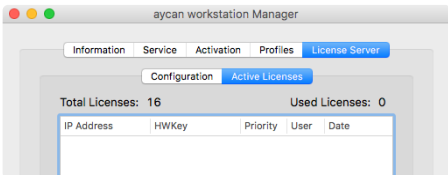The tab “Active Licenses” of the License Server (optional) displays the status of the applied licenses in the network.

“Total Licenses” displays the total number of available authorizations, “Used Licenses” the number of the actual available authorizations. In the list below, all licenses which are currently in use are displayed as well as their “IP Address”, “Hardware Key”, “Priority”, “Date” (and time of when the connection was established), “User”. You can arrange the columns to your liking via Drag & Drop .

Manager . Activation / License Server (optional)

If a License Server is set up (see above), a client can enter the IP address of the server in the lower section of the “Activation” tab and start the connection by clicking on “Activate license client” .

If the client ends **aycan workstation** and the **aycan workstation Manager**, the previously used license is released at that time and is immediately available for other clients.